
FLOCARE® GRAVITY GIVING SET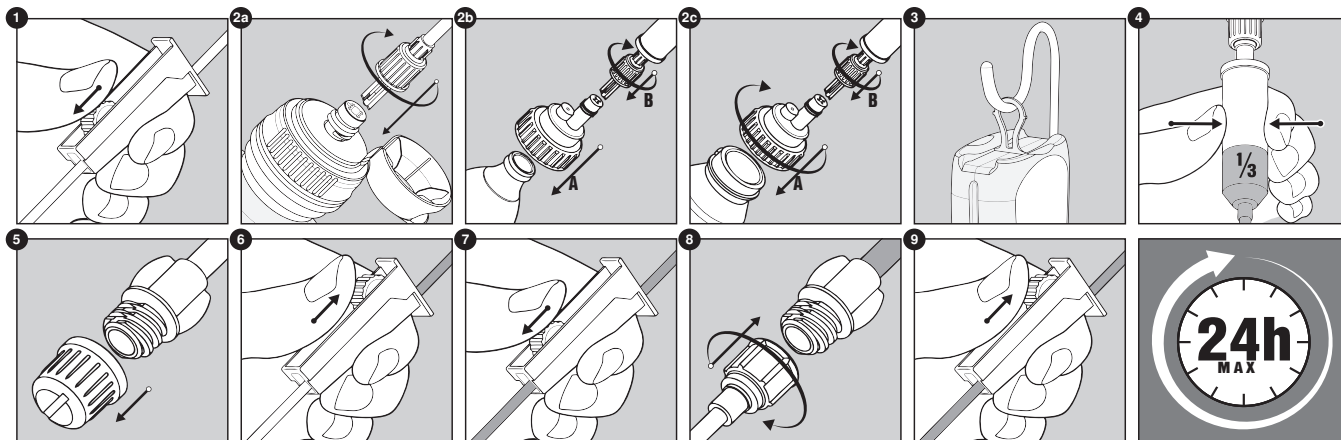

Instructions for use / Gebrauchsanleitung / Gebruiksaanwijzing / Mode d'emploi / Instrucciones de uso / Istruzioni per l'uso / Instruções de utilização / Brugsanvisning / Bruksanvisning / Käyttöohje / Bruksanvisning / Οδηγίες Χρήσης / Kullannin Käyttöohje / Instrukcja używania / Návod k použití / Návod na použití / Használati utasítás / Naudojimo instrukcija / Lietošanas instrukcija / Kasutamishuised / تعليمات الاستخدام

EN • FLOCARE® GRAVITY GIVING SET
 Enteral giving set for gravity tube-feeding intended to deliver tube feed for patients who tolerate inconsistencies in flowrate.
WARNINGS: For enteral use only / Choking hazard; loose parts / Single patient use only, do not use for more than 24 hours / Do not reuse as this may create a risk of infection and cause the product not to function as it should / Do not use with an enteral feeding pump / No modification of the devices is allowed. **PRECAUTIONS:** Store in a dry, clean place / Consult your healthcare professional before using the device / Notice: any serious incident that has occurred in relation to this device is to be reported to Nutricia and the local competent authority / The giving set is only to be connected with ENFit enteral devices and enteral feeding reservoirs. **SET UP:** Close the roller clamp (1), connect the giving set to the feeding reservoir (2) and hang the feeding reservoir at a meter height from the stomach (3). Gently squeeze the drip chamber until it has filled to a third full (4), remove protective cap (5), open roller clamp (6) until giving set is primed and close clamp again (7). **USE:** Connect the set to the enteral feeding tube (8). Control the speed/rate of feeding by slowly opening or closing the roller clamp (9). Every 20 drips equals about 1mL. Regularly check the feeding container to ensure the flow rate is correct. Always monitor the patient during feeding. After use, dispose of product and packaging in accordance with hospital, administrative and/or local policy.

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