
FLOCARE® INFINITY™ GIVING SET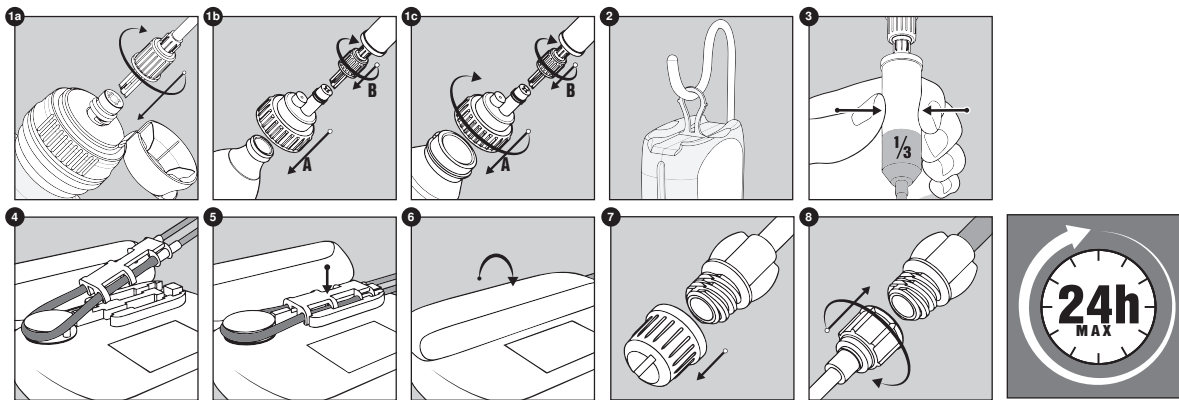

Instructions for use / Gebrauchsanleitung / Gebruiksaanwijzing / Mode d'emploi / Instrucciones de uso / Istruzioni per l'uso / Instruções de utilização / Brugsanvisning / Bruksanvisning / Käyttöohje / Bruksanvisning / Οδηγίες Χρήσης / Kullanım Kılavuzu / Instrukcja używania / Návod k použití / Návod na použití / Használati utasítás / Naudojimo instrukcija / Lietošanas instrukcija / Kasutamishuised / تعليمات الاستخدام

EN • FLOCARE® INFINITY™ GIVING SET - Enteral giving set for FloCare Infinity pump assisted feeding. **WARNINGS:** For enteral use only / Choking hazard: loose parts / Single patient use only, do not use for more than 24 hours / Do not reuse as this may create a risk of infection and cause the product not to function as it should / No modification of the devices is allowed. **PRECAUTIONS:** Only use in conjunction with a FloCare Infinity enteral feeding pump / Store in a dry, clean place / Consult your healthcare professional before using the device / Consult the FloCare Infinity pump instructions for use for operating of the feeding pump and setting up of the feeding system / Notice: any serious incident that has occurred in relation to this device is to be reported to Nutricia and the local competent authority / The giving set is only to be connected with ENFit enteral devices and enteral feeding reservoirs. **SET UP:** Connect the giving set to the feeding reservoir (1) and hang the feeding reservoir onto e.g. a pole (2). If present, gently squeeze the drip chamber until it is filled to a third full (3). Open the pump door and position the looped section of the giving set tubing around the pump rotor (4), stretching lightly, seat the cassette into the pump (5) and close the door (6). Remove protective cap (7) and fill the pump set completely as described in the instructions for use manual of the Infinity pump. **USE:** Connect the giving set to the enteral feeding tube (8). Program the feeding pump following the instructions of the FloCare Infinity user manual. Regularly check the feeding therapy to ensure the flow rate is correct and the enteral feeding system is operating normally. Always monitor the patient during feeding. After use, dispose of product and packaging in accordance with hospital, administrative and/or local policy.

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Production location: Nutricia Pharmaceutical Wuxi Co., Ltd., P.R. China