Flocare® Bolus Adaptor

REF 690390

MD

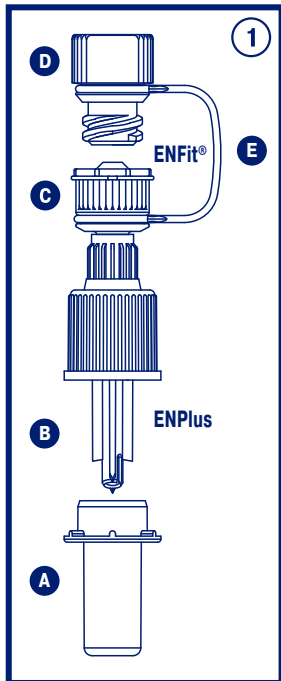
STERILE EO

STERILIZE

PHT
DEHP

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EN: Flocare® Bolus Adaptor. This device is intended to connect flexible enteral nutrition containers, equipped with ENPlus port, to ENFit® syringes for bolus feeding. Intended for enterally fed patients. For use by healthcare professionals or by lay users and patients after proper training. **⚠️ Warnings:** For enteral use only. **⚠️** NOT for intravenous use. Do NOT use for more than 24 hours. Do NOT use if labelling is incomplete or illegible. Always ensure appropriate patient monitoring. Small parts are a potential choking hazard if this device is not used as intended. Use under adult supervision. In case of drug administration to the patient, always verify that the medication delivered by the adaptor is suitable for enteral feeding. Any reprocessing (especially re-sterilization) will compromise the structural integrity of the device. The device may become a source of infection and have altered performance after use and in case of reuse. **Important notes:** Ready to use. **⚠️ DEHP:** not intentionally added during the manufacturing process. Not formulated with BPA or natural rubber latex. Follow good clinical practice and all local and national regulations for transportation and disposal. Immediately report serious incident to the Manufacturer and to the Competent Authority in your country. **⚠️** This Instruction for Use is available at www.cedicbio.com.

Instruction for Use: 1. Wash hands and adopt good hygiene practice. 2. Confirm the container is an enteral feeding reservoir. Do not connect the adaptor to non-enteral feeding containers. 3. Remove the cap (if applicable) from the enteral feeding container. 4. Remove cap from the ENPlus proximal end. 5. Firmly screw the ENPlus proximal end into the cross port of the enteral feeding container. 6. Remove ENFit® female cap from ENFit® port. 7. Remove the cap (if applicable) from the ENFit® syringe. 8. Securely screw the ENFit® syringe into the ENFit® male port (see drawing 2). 9. Turn the container upside down by holding firmly the entire unit to withdraw the liquid through the syringe (see drawing 3). 10. Repeat step 9 as needed, without disconnecting the adaptor from the container between subsequent withdrawal. Disconnect and discard the adaptor no later than 24 hours after initiating the use. Refer to syringe instruction for use for details on its indications and duration of use.

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